


# PROTECT YOUR PATIENT



## DISKCOVER SYSTEM USE

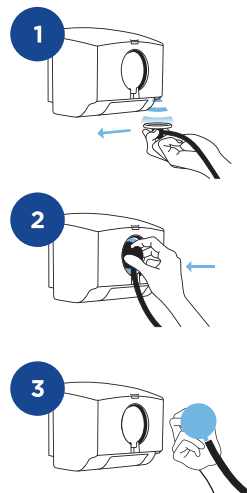

## MAINTENANCE

### Clean Cassette® Indicator

Blinking **orange light**

Low, replace soon



Blinking **red light** or **red light** ring  
Empty, replace immediately



### Battery Light Indicator

Blinking **red light**

Low, replace immediately



**No light** and system unresponsive  
Depleted, replace immediately

## TROUBLESHOOTING

### Clearing a Dispense Cycle Error:

1. Wave 1x to clear error, wait 6 seconds.
2. Repeat 1x, if necessary.
3. If error persists, briefly remove and reinstall Clean Cassette.  
Wave 1x to clear error.
4. If error persists, insert a new Clean Cassette.
5. Call Customer Support.



Instructions for Use video:  
Scan with camera phone, or visit  
[DiskCover.com/DiskCover](https://DiskCover.com/DiskCover)  
**Customer Support:**  
[Support@AseptiScope.com](mailto:Support@AseptiScope.com)

See reverse for more information.

**Customer Support: 1-844-980-2998**

© 2021 AseptiScope, Inc. All rights reserved. AseptiScope, Clean Cassette, and related logos are registered trademarks of AseptiScope, Inc. DiskCover and related logos are trademarks of AseptiScope, Inc. LBL-1005-02



REF DSA-L1