

THE **DISKCOVER**[™]
SYSTEM
PROTECT YOUR PATIENT

Instructions for Use

Table of Contents

Introduction	3
Intended Use.....	3
Warnings and Precautions.....	4
DiskCover System Components and Accessories.....	4
Description.....	5
Install/Replace Clean Cassette.....	6
DiskCover System Use.....	7
Clean/Disinfect Dispenser.....	8
Dispenser Installation	8
Install/Replace Batteries	9
Dispenser Replacement.....	9
Clean Cassette Disposal	10
Light Indicators for Normal Operation	10
Troubleshooting.....	11
Specifications	13
Symbols.....	14
Warranty	15
Support	15

THE DISKCOVER™ SYSTEM

Introduction

AseptiScope™ disk covers have been proven effective in providing an aseptic stethoscope barrier to reduce potential pathogen transmission.¹ The AseptiScope™ DiskCover™ System automates the touch-free application of AseptiScope disk covers to stethoscope diaphragms to ensure hygienic patient contact. The DiskCover System includes a wall-mounted dispenser that houses a Clean Cassette™ containing single-use disk covers. Once applied, the disk cover helps prevent pathogen transmission¹ without compromising the sound quality² of the stethoscope. The DiskCover System requires no formal training, takes seconds to apply an aseptic barrier, and provides clinicians a clean stethoscope surface for immediate use.

Intended Use

The DiskCover System automates the touch-free application of single-use disk covers to stethoscope diaphragms as an aseptic barrier for hygienic patient contact to prevent pathogen transmission. The DiskCover System is intended for use in all clinical settings.



Warnings and Precautions

- Read all instructions prior to use.
- For use on intact skin only. Do not place the disk cover over open wounds.
- Do not install in location where the dispenser is exposed to extreme cold or hot conditions or in wet environments such as showers.
- Do not install in areas with less than 7" clearance above and below the wall mount.
- Do not install without using a level.
- Do not use the dispenser or Clean Cassette if there is visible physical damage.
- Disk covers are single use only. Do not reuse.
- Only use Clean Cassettes provided by AseptiScope.
- Install only Alkaline C batteries.
- Dispose of used Clean Cassettes in accordance with federal, state or local regulations.
- Do not use disinfectants containing hydrogen peroxide or other abrasive agents on the dispenser as it may cause damage.
- USB port for service technician use only.
- Do not use excessive force when applying the disk cover.

DiskCover System Components and Accessories

DiskCover Dispenser, with 3-Alkaline C batteries [\[REF\]](#) DSA-1


DiskCover Clean Cassette, containing roll of DiskCover Disks [\[REF\]](#) CSA-1

Wall Mount [\[REF\]](#) WMS-01


Placard [\[REF\]](#) DSA-L1

Description


Dispenser - Front

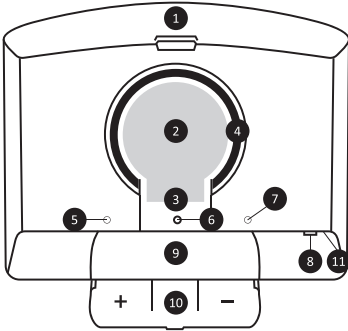


Dispenser - Back

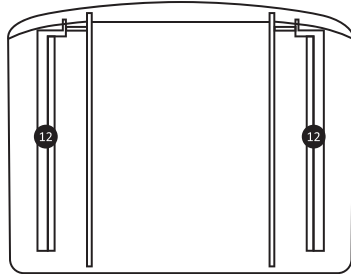

Clean Cassette - Front

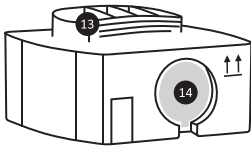


Clean Cassette - Bottom

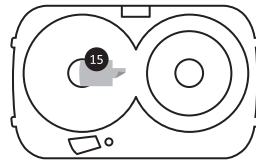

DiskCover Disk (disk cover)



Wall Mount

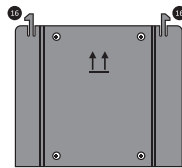

1. Dispenser lid
2. Application window
3. Stethoscope tube notch
4. Light ring
5. Clean Cassette indicator light
6. Safety sensor*
7. Batteries indicator light
8. Activation sensor

9. Battery compartment
10. Battery door
11. USB port
12. Mounting rails
13. Clean Cassette handle
14. Disk cover window
15. Shipping insert
16. Release tabs

*Senses objects that could potentially disrupt the proper advancement and positioning of a new disk cover in the application window. If the sensor is triggered and disk cover halted before being fully placed in the application window, see DiskCover System Use for activation steps.

Install/Replace Clean Cassette

1. Remove Clean Cassette from protective bag (Figure 1).
2. Remove shipping insert from bottom of cassette (Figure 2).
3. Open dispenser lid.
4. If replacing, lift the used Clean Cassette by its handle and remove.
5. Orient the new Clean Cassette this side up, as indicated by up arrow, with disk cover window facing toward you (Figure 3).
6. Holding the Clean Cassette handle, gently lower Clean Cassette into dispenser (Figure 4).
7. Close dispenser lid.
8. If a Clean Cassette has been properly installed, a green Clean Cassette indicator light will briefly illuminate (Figure 5).

Figure 1

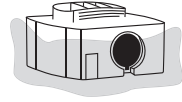


Figure 2

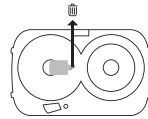


Figure 3

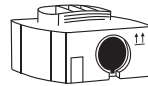


Figure 4


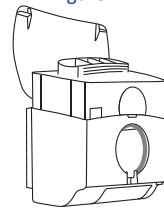


Figure 5

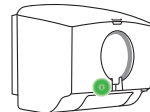

DiskCover System Use

1. **WAVE:** Grasp the back of the diaphragm and wave under the touch-free sensor (right side of the dispenser) (Figure 6).
2. **APPLY:** After light ring turns green, firmly place the stethoscope diaphragm into the application window. (Be sure stethoscope tubing fits within the notch) (Figure 7).
3. **REMOVE:** Withdraw the stethoscope – a DiskCover Disk is now applied! (Figure 8)
4. After patient exam, pull the white tab of the disk cover away from the stethoscope and dispose of properly (Figure 9).

IMPORTANT: A Clean Cassette must be installed to activate the dispenser.

Figure 6

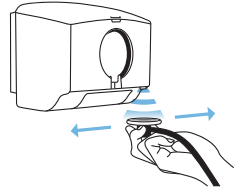


Figure 7


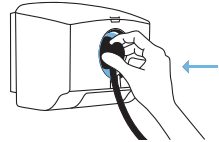


Figure 8



Figure 9

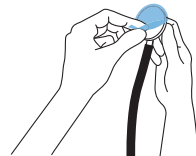

Visit DiskCover.com for virtual Instructions for Use and more information.

Clean/Disinfect Dispenser

The dispenser can be cleaned with mild soap and damp cloth or with non-abrasive disinfecting agents.

Dispenser Installation

1. Properly orient wall mount as indicated by up arrows facing outward, ensuring horizontal edge is level and with 7" clearance above and below the wall mount (Figure 10).
2. Secure the wall mount to the wall with appropriate screws (Figure 10).
3. Position the dispenser above the wall mount, align with the rails and slide down onto the edges of the wall mount (Figure 11) until the dispenser clicks into place (Figure 12).
4. Install batteries (see Install/Replace Batteries section).
5. Install Clean Cassette (see Install/Replace Clean Cassette section).

Figure 10

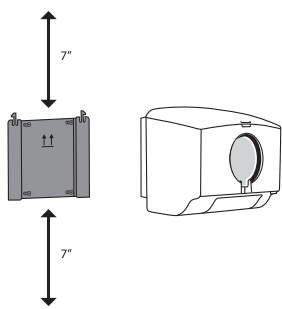


Figure 11


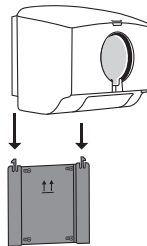


Figure 12

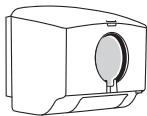

Install/Replace Batteries

1. Open battery door
2. If replacing batteries, remove depleted batteries and dispose appropriately.
3. Insert three (3) C batteries as shown on the inside battery door, placing center battery in last (Figure 13).
4. Close battery door. If properly installed, the batteries indicator light will briefly appear green. Dispenser will be ready for activation after 3 seconds (Figure 14).

Figure 13

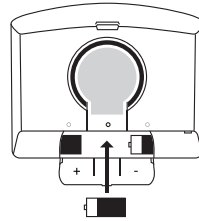


Figure 14



Dispenser Replacement

1. Remove Clean Cassette and batteries.
2. While holding the sides of the dispenser, locate the release tabs near the top of the wall mount (figure 15) and press towards the center (Figure 16).
3. While depressing the tabs, raise the dispenser until clear of the wall mount (Figure 17).
4. If replacing the dispenser, see *Dispenser Installation* section.

Figure 15

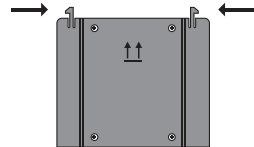


Figure 16


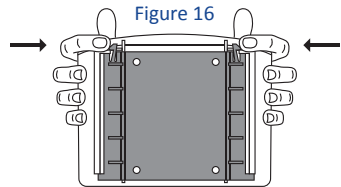


Figure 17

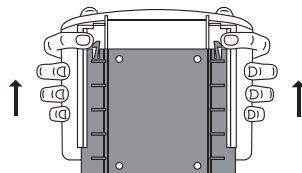

Clean Cassette Disposal

The Clean Cassette is made from recyclable plastics but contains other materials that cannot be recycled. AseptiScope offers a Clean Cassette recycling service. Contact customer service for details.

Light Indicators for Normal Operation

Illuminating green light ring (in sequence)	Wait, disk being advanced into position
Solid green light ring (briefly illuminated)	Disk cover is in position and ready for application (Note: the light ring will not remain green, however, a disk cover can be applied whenever a disk cover is fully visible in the application window.)
Blinking red Clean Cassette indicator light	<15 disk covers remaining in the Clean Cassette, replace Clean Cassette immediately to prevent disruption of dispenser use
Blinking red batteries indicator light	<10% of battery life remaining, replace 3-Alkaline C batteries immediately to prevent disruption of dispenser use
Solid green Clean Cassette indicator light (briefly illuminated)	An authenticated Clean Cassette has been installed and dispenser is ready for use.
Solid green batteries indicator light (briefly illuminated)	Batteries installed properly

Troubleshooting

<p>Disk cover did not adhere to diaphragm</p>	<p>Be sure to hold the back of the stethoscope diaphragm (do not hold by the tubing) and firmly place the stethoscope diaphragm into the application window. Be sure stethoscope tubing fits within the notch. Do not use excessive force. If application is still unsuccessful, manually remove by pulling disk cover from application window and discard. Wave below the activation sensor to position new disk cover in application window and reattempt.</p> <p>TIP: Not all stethoscopes are made equal! Lifting an edge of the stethoscope first may improve disk cover adherence.</p>
<p>Disk is not fully exposed in application window</p>	<ul style="list-style-type: none"> • If light ring is red, Clean Cassette is jammed. Try removing any visible disk covers and wave below the activation sensor after 3 seconds. If dispenser does not activate/advance disk cover, check for and remedy any indicator lights. • If light ring is not red and dispenser is operational, Clean Cassette is not jammed. A partially exposed disk cover may be visible, but should not alter performance.
<p>Activation sensor is not activating dispenser</p>	<p>NOTE: The dispenser will automatically pause 3 seconds between each activation.</p> <p>Ensure a Clean Cassette is installed. Check the batteries and Clean Cassette indicator lights. Ensure nothing is blocking the activation sensor. Ensure nothing is covering the safety sensor. Replace batteries, as necessary.</p>

<p>The dispenser will not activate despite working batteries and a Clean Cassette is installed with disk covers present, no light indicators are present</p>	<p>Perform system reboot:</p> <ol style="list-style-type: none"> 1. Remove Clean Cassette and batteries. 2. Wait 5 seconds. 3. Reinstall batteries. <p>If green battery indicator light appears, ensure shipping insert is removed and reinstall Clean Cassette.</p> <p>If no green Clean Cassette indicator light appears, contact Customer Service.</p>
<p>Alternating yellow, green, and/or red cassette and battery indicator lights</p>	<p>Perform system reboot:</p> <ol style="list-style-type: none"> 1. Remove Clean Cassette and batteries. 2. Wait 5 seconds. 3. Reinstall batteries. <p>If green battery indicator light appears, reinstall Clean Cassette.</p>
<p>Solid red light ring</p>	<p>Disk cover may be jammed in Clean Cassette.</p> <ol style="list-style-type: none"> 1. Remove Clean Cassette. 2. Remove any visibly jammed disk covers from the backing tape. If no jammed disk covers are visible, perform system reboot (described above). 3. Reinstall Clean Cassette <p>If light ring again turns red, replace with new Clean Cassette.</p>
<p>Solid Red Clean Cassette indicator light (briefly illuminated)</p>	<p>Clean Cassette failed authentication. Check to ensure the shipping insert has been removed.</p> <p>Perform system reboot:</p> <ol style="list-style-type: none"> 1. Remove Clean Cassette and batteries. 2. Wait 5 seconds. 3. Reinstall batteries. <p>If green battery indicator light appears, reinstall Clean Cassette.</p> <p>If no green Clean Cassette indicator light appears, contact Customer Service.</p>

No lights	<p>Ensure the shipping insert has been removed from bottom of Clean Cassette.</p> <p>Ensure Clean Cassette is installed. If a Clean Cassette is not installed properly, no lights will illuminate.</p> <p>If after installing a Clean Cassette, no lights appear replace 3-Alkaline C batteries.</p>
-----------	--

Specifications

Dispenser:

- Dimensions: 6.01" x 7.72" x 6.23"
- Weight: 1.43 lbs
- Weight (with Clean Cassette): 2.75 lbs
- Waterproof Rating: IP22. Protection from dripping water at a 15-degree angle from proper installation.
- Battery: 3-Alkaline C-Cells
- Useful Life: 2 years

Clean Cassette:




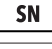





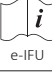




- Dimensions: 4.96" x 7.13" x 4.20"
- Weight: 1.32 lbs





Wall Mount:

- Dimensions: 5.00" x 5.90" x 0.35"
- Weight: 0.95 lbs

For additional specifications, visit www.DiskCover.com

Symbols

Symbol	Definition
	Manufacturer
	Date of Manufacture
	Lot Number
	Serial Number
	Catalog Number
	Do not use if package is damaged
	Keep dry
	Temperature Limits
	Humidity Limits
	Consult instructions for use
	Does not contain latex
	Product contains electrical / electronic components. Dispose / recycle appropriately for electronic devices.
	Caution
	Do not re-use

Symbol	Definition
	Battery indicator
	Activation sensor
	Replace Clean Cassette indicator
	RoHS

Warranty

AseptiScope warrants the dispenser and Clean Cassette shall be free from defects of workmanship and materials for a period of one (1) year from the date of purchase (the “warranty period”). If, during the warranty period, the dispenser or Clean Cassette fails under normal use and a replacement authorization number is obtained from Technical Support, the manufacturer will replace the dispenser at no cost. This warranty does not cover damage caused by use inconsistent with product labeling, accident, abuse, negligence, alteration, and other defects not related to materials or workmanship. Warranty is voided if the dispenser is repaired or altered by anyone other than the manufacturer.

Support

To order additional DiskCover Systems or replacement Clean Cassettes, please contact Customer Service, your sales rep, and/or order on-line at DiskCover.com.

The dispenser and Clean Cassette are not designed or warranted for field service. Please contact Customer Service: 1-844-980-2998.

References:

1. Vasudevan R, Shin JH, Chopyk J, et al. Aseptic Barriers Allow a Clean Contact for Contaminated Stethoscope Diaphragms. *Mayo Clin Proc Innov Qual Outcomes*. 2020;4(1):21–30.
2. Data on file.

Patent pending.

© 2020 AseptiScope, Inc. All rights reserved. DiskCover, Clean Cassette and AseptiScope and related logos are trademarks of AseptiScope, Inc.

Learn More at DiskCover.com



AseptiScope.com

San Diego, CA 92103 USA
Customer Support
1-844-980-2998