

THE DISKCOVER™ SYSTEM

How-to-videos and additional support:
Scan with camera phone, or visit
DiskCover.com/Support



AseptiScope.com
San Diego, CA 92103 USA
Customer Support: 1-844-980-2998
support@AseptiScope.com

© 2021 AseptiScope, Inc. All rights reserved. AseptiScope, Clean Cassette, and related logos are registered trademarks of AseptiScope, Inc. DiskCover and related logos are trademarks of AseptiScope, Inc. LBL-1053-01

NEW USERS START HERE

THE DISKCOVER™ SYSTEM

from  AseptiScope®


QUICK START GUIDE



DiskCover Dispenser: [REF](#) DSA-1
DiskCover Clean Cassette*: [REF](#) CSA-1
Wall Mount: [REF](#) WMS-01
Informational Placard: [REF](#) DSA-L1
Each component sold separately


DISPENSER, CLEAN CASSETTE, AND PLACARD INSTALLATION

1 Dispenser: Wall Mounting

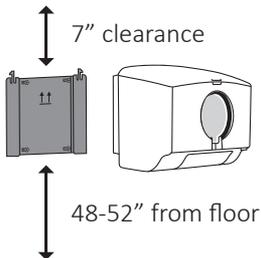


Recommend installing close to the point of patient care


2 Dispenser: Insert Batteries

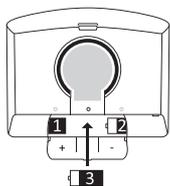


3 Clean Cassette: Remove Shipping Insert

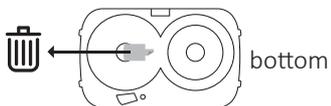

4 Clean Cassette: Insert into Dispenser

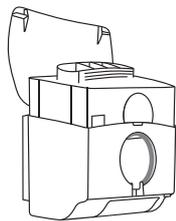

5 Insert Informational Placard

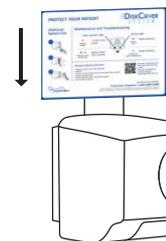

Additional information is found under the dispenser lid




Installation video:
Scan with camera phone,
or visit [Vimeo.com/444342458](https://www.vimeo.com/444342458)

DISKCOVER SYSTEM USE


WAVE

Wave under touch-free sensor (at lower right side)

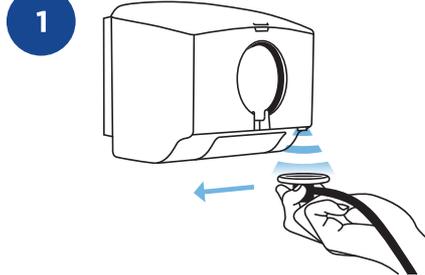

APPLY

Press firmly after light ring turns green

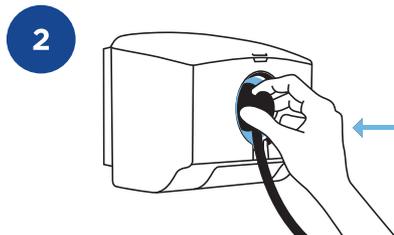

REMOVE

Withdraw with disk cover applied

